

ATTACHING A REGULATOR TO AN OXYGEN TANK INSTRUCTIONS

1



Note the two "teeth" on the inside of the regulator.

2



The "teeth" will fit into the matching two holes on the neck of the oxygen tank.

3



Carefully fit the "teeth" into the neck, and tighten the regulator using the fastener.

4



Make sure the regulator is on straight, and not crooked like in this picture.

5



Once the regulator is on, use the wrench to open the tank (turn to the left to open). Some tanks have a wrench built in. You can twist that to the left to open the tank.

6



You can tell how much air in the tank based on where the needle is pointing on the gauge.

When you are done using the tank, close the tank by turning the wrench to the right.