

**How to Access the QR+ Feature on the Luna G3 Pap**


3B Luna QR is a free smartphone app that allows users to see pap therapy data on their phone in real time. Easy to use, and compatible with android and apple smartphones.

**Step One:**

- Download and install the 3B Luna QR app (enabling camera permissions)
- 3B Luna QR: [Google Play](#) or 3B Luna QR: [Apple App store](#)

**Step Two:**

Access Luna G3 “Report” screen by scrolling to report, then click dial to access

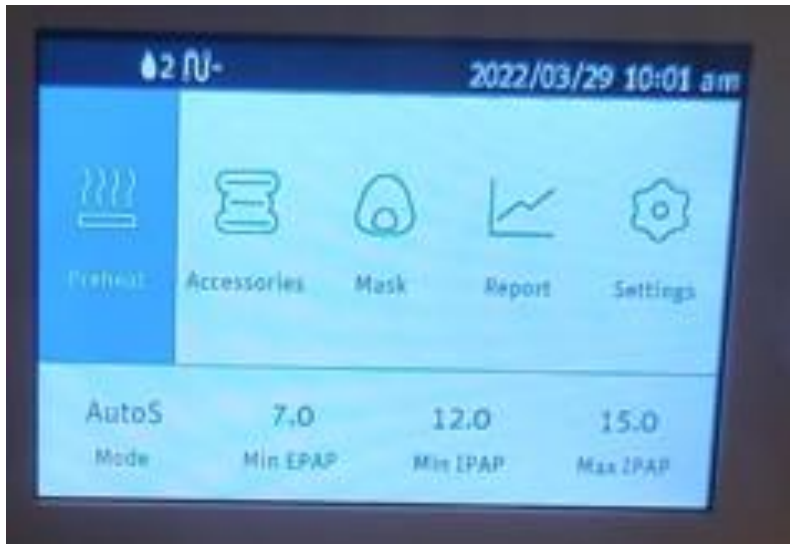

**Step Three:**

Turn and scroll to select iCode QR/QR+



**Step Four:**

Turn dial to the right to access iCode QR+



**Step Five:**

Open app and select “iCode QR & iCode QR Plus” then scan QR code.

